


Versare® Medi-Wall Divider

Assembly Instructions

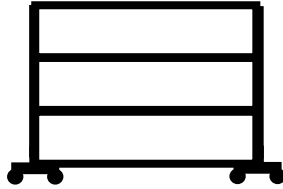

1. Remove window and lay panels flat on the ground.
 - a. For panels that have film covering, remove this before proceeding. Pull film from frame edge in small sections at a time. You can also carefully cut the film, with a box cutter, along the frame and peel it from the window panel material.
2. Sleeve the side trim bar into the channel on the short side of the panels, starting with one panel at a time. Do this with the product flat on the ground.
 - a. Repeat this on the other side of the panel with the second trim bar.
3. Arrange the next panel under the first and continue sliding the side trim bar into the channel on this next panel.
 - a. The side trim effectively locks and holds the panels in place.
 - b. Repeat this again if assembling 3-panel models.
4. Once the entire side trim has been pushed completely onto the panels, now grab the foot assembly and slide it on open section of channel on the last panel.
 - a. Repeat on both sides.
5. If your model does not have wheels, you can lift up in to position.
6. For models with wheel, lock each caster before lifting up in to position.

